


Application

Universal fixation for tubes and lines especially CVC




1

Remove the backing paper and stick the holder on a suitable place.

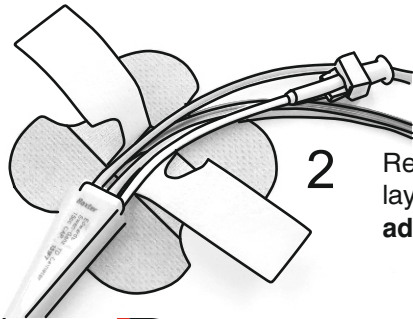

2

Remove cover from Velcro strip, lay "tubes" / lines into the adhesive area .

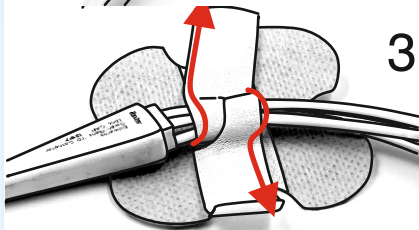

3

Close the wings and press lightly.



For fastening flat filters and other components- Simply press the filter onto the adhesive surface.

The patient can be washed or showered, the fixation **holds even if it gets wet.**
The fixation can be opened and closed several times.

Unifix




ord. no. 8501500

Unifix Hydrocolloid



ord.no.8501509

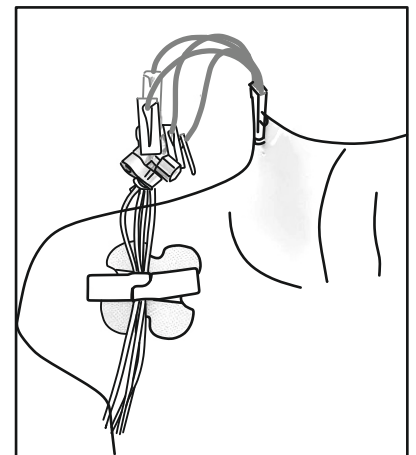

Duration depending on use up to 3 days
Hydrocoll. - 6 days

Most parts of the SECUTAPE® system can be combined with each other, Secutate fixations are reliable and offer the best possible hold on the skin. They can be used for several days.