


Application

Standard fixation size 2/3


for the fixation of:

- PEG
- CAPD
- ZVK
- Suprapubic catheters
- Infusions / NFC
- Urin-catheter
- Redondrains
- Ports
- Wounddrains
- Probes for pressure monitoring
- and all other tubes and wires.

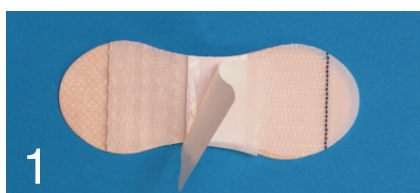


1

Prepare the skin clean and free of grease.
Take the back side liner off and apply the fixation on a suitable place.

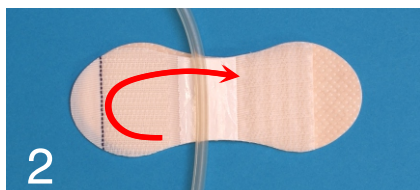

2

Take the paper of the flap and put the tube on the adhesive line.

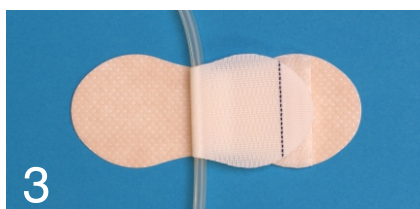

3

Close the velcroflap with a little pressure.


Fixation can be opened and closed several times.

Standard fixation

size 2+
ord.no.
8500201



size 3+
ord.no.
8500301

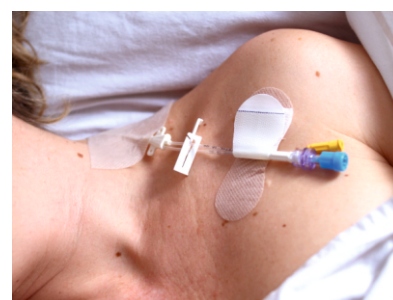

Depending on the application, it lasts up to 5 days,

The tube can be taken out and fixed again for several times.

The fixation lasts even if it gets wet !

For heavy tubes use size 3+

Most parts of the SECUTAPE® system can be combined with each other,
Secutape fixations are reliable and offer the best possible hold on the skin.
They can be used for several days.



TechniMed AG
SWITZERLAND
since 1986

Kirchstrasse 54
9400 Rorschach
info@technimed.ch

Switzerland
fon 0041/71 844 49 20
fax 0041/71 845 52 45

Germany
fon 0800 007 98 92
fax 0800 941 30 12

Austria
fon 01/440 79 79
fax 01/440 79 90

www.secutape.ch

CE
Made in Germany
© TechniMed AG